Pump Kit Cleaning Breastfeeding Hygiene & Safety

Guidelines for Mother and Baby

For over 25 years,

Medela® has set the standards of excellence in our field and by doing so...raised the bar for the rest of the industry. At Medela, safety comes first. Below are general guidelines to ensure the safety and effectiveness of the pumping experience for you and your baby.

Hygiene during Pumping


Good hygiene is necessary if you plan to give your expressed milk to your baby. Before expressing your milk, you should wash your hands well with soap and water. Wash all the collecting bottles and breastpump parts that touch your breasts or the milk. Use hot, soapy water or a dishwasher.

First, you must take apart the pump kit.

1. Remove tubing from pump and breastshield (if your pump has tubing).


2. Remove bottle(s) from breastshield(s).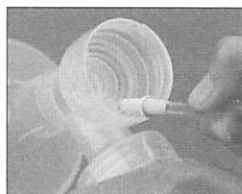

3. Remove the white membrane from the valve and the valve from the breastshield.

4. Separate breastshield if using the PersonalFit™ Breastshield that has two pieces.

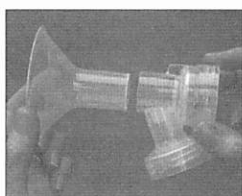5. Wash all parts in hot, soapy water and rinse with clear water.
6. Do not store wet or damp parts; allow parts to air dry on a clean towel. If your baby is premature or ill, strictly follow the hospital guidelines for sterilization of your pumping kit.
7. Sanitize your kit with boiling water before first use (if not using a sterilized kit), and clean per your kits instructions.

Condensation Concerns

Condensation is a natural result of humidity. If condensation occurs you can eliminate it by allowing your tubing to dry completely after pumping.

- When your pumping session is over, while the pump is still running, remove the tubing from the back of the breastshield.
- Continue to let the pump run with only the tubing attached.
- Remove the shield from your breast and cap and store your milk.
- Clean breastshields, valve and membrane as instructed.
- Continue to let the pump run with the tubing attached.
- After pump parts are cleaned, return to pump and turn it off.
- **Allowing the pump to run while you are attending to your milk and cleaning your pump parts will remove any condensation that may have occurred during your pumping session.**

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Medela is a registered trademark and PersonalFit is a trademark of Medela, Inc.
© 2007 Medela, Inc. All rights reserved.
Medela es una marca registrada de Medela Inc. PersonalFit es una marca comercial de Medela, Inc.
Medela, Inc., P.O. Box 660, McHenry, IL 60051-0660 USA